



HemaTrax[®]-CT 3.4.0 Release Notes

General Changes:

- Product code database updated to ICCBBA version 7.4.0, released May 2017.
- When using the Get Data button from print log, it was calculating UTC time when 10 year option selected. It now will default to 23:59. Exactly as it would if you selected the 10 year expiration button option.
- Manual option for Daylight Savings Time/Standard Time added. Checkbox enables this feature.
- Fixed a double logout recorded in the Activity Log when selecting the Exit button.
- Fixed a situation where the Special Message would disappear after selecting it, rendering the label invalid to print.
- Added search option by DIN for print log.
- Changes made to the ICCBBA product code database Attributes to more accurately display certain text:

“T+B Cell Reduced” now displays “T/B Cell Reduced”

“Alpha Beta T+B Cell Reduced” now displays “Alpha Beta T/B Cell Reduced”

“NS HES + NS DMSO Present” now displays “HES + DMSO Present”

“Viral Specific T Cells Enriched” now displays “Viral Specific T Cells Enriched

“T Reg Enriched” now displays “T Reg Enriched”

HemaTrax[®]-CT version 3.4.0 does not impact existing user configuration, printer, and scanner set- up