



HemaTrax CT

3.5.0 Release Notes

- Product code database updated to ICCBBA version 7.19.0, released November 2018.
- Added GRID support. If a GRID number is scanned or entered in the Donor ID field, the software will recognize the GRID number and display that on the 4x4 label with associated barcode.
- New label format. The Cord Blood Segment Set format is a label set that prints a 1.5"x2" label with several smaller DIN labels. You can order the new labels by contacting 1-800-356-6126 and using the part number CRYO-CBA-CG-KT.
- Removed duplicate donor information for autologous labels. If you select an autologous donation type, the text on the label will now only display in the lower right quadrant. Previously, it would display in the upper and lower right quadrants.
- Changed the way products with an anticoagulant of "Not Specified" display their core conditions (volume text). When a not specified product is selected, you may select an anticoagulant (Heparin, Citrate, or Heparin+Citrate). If one of the anticoagulants are selected, now the core conditions will reflect the volume information for that anticoagulant.
- Added additional product type option of Investigational Device. When selected, the appropriate text will display in the lower left corner of the label.
- Corrected a security permissions issue involving the auto logout feature. Under certain conditions, the software would auto logout and log back into a screen that a lesser user should not have access to.