



HemaTrax CT

3.6.0 Release Notes

1/2020

- Product code database updated to ICCBBA version 7.31.0.
- In an effort to improve security, we have moved the DIN Setup dropdown from the Label Design page to the User Management page. This requires an Admin to change any of the Generate DIN settings.
- Additionally, when you select the Generate DIN button, it will now check the print log to make sure that DIN has not been generated before. If it has, you will get a warning message and not be allowed to use it. NOTE: This only affects DINs that are generated from the Generate DIN button. It will not check for duplicates on scanned, manually entered, or DINs pulled from the print log.
- Added support for the Single European Code. SEC setup and options are now available in the User Management page.
- Fixed an issue where the GRID barcode would not appear if the following steps occurred:
 - a) A label size other than the 4x4 was selected.
 - b) The user navigates to the Print Log, selects a record and presses the Get Data Button.
 - c) On the Label Design tab, the label size was changed back to the 4x4 label
- Changed the drop-down field “Donation Type” to “Collection Type”.
- Improved alignment of human readable text below the barcodes.
- Included the option to print a blank GRID: _____ line by entering 00000000000000000001 for the Donor ID.